SORDINA

tecnologie per la vita / *technology serving life*

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The most advanced technology for operating tables

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MT System is the name of the most advanced operating system with table top transfer exclusively designed by Sordina.

This innovative table could only be designed by a company with over a hundred years' experience.

In fact, Sordina was founded way back in 1880, and it was the first company in the world to design a mechanically controlled operating table.

On the strength of the know-how they have built up in more than a century, Sordina has always focused on excellence: advanced research, close cooperation with the most important modern surgical units, the exclusive use of cutting-edge technology and materials.

The MT System comprises two types of interchangeable surgical table tops and three models of column bases: transferable, on wheels and fixed to the floor. There are also three models of trolleys.

Advanced, practical and highly ergonomic, MT System offers a virtually endless range of positions for the patient and numerous different configurations, making it ideal for absolutely any kind of surgery.

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The surgical table tops are divided into modular and interchangeable sections, made of a special ultra light and highly resistant aluminium alloy. This material gives the operating table top excellent mechanical strength, resistance to corrosion and reduced weight assuring a much easier transferring of the table top. Manufactured without any welding, jutting edges or bulky mechanical parts, the surfaces are extremely smooth to make cleaning and disinfection easier. The electro-hydraulic movements make the system more solid, without any endplay and with very smooth and precise movements compared to electromechanical systems.

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MT8 operating table top.

The structure is made of a special highly resistant stainless alloy. It is divided in between 6 and 11 modular sections, depending on the required configuration. It offers considerable advantages: the upper back section can be extended with no need for extra tables, and the upper and lower back sections can be raised and lowered automatically and separately. Various models of automatically moved legs sections. Automatic lengthways translation of 305 mm.

Different pads can be fitted to the table top as required.

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MT6 operating table top.

The surgical table tops are made of stainless alloy. It is divided in between 4 and 7 modular sections, depending on the required configuration. It has a back section and various models of legs sections and head sections, with gas spring assisted movements.

Lengthways translation of 305 mm to make the operating table completely radiotranslucent. Different pads can be fitted to the table top as required.

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Extremely practical even in confined spaces, the MT System column bases are the result of a very precise ergonomic research made to facilitate the medical staff in their movements and the use of the medical equipment.

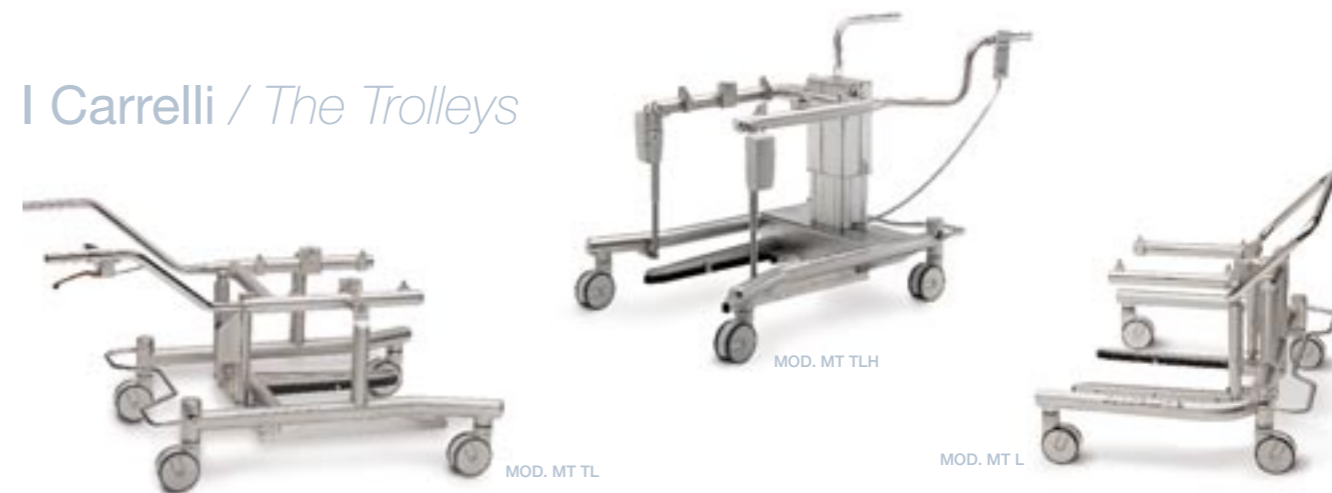
Three models are available:

- *A flat stainless steel base which is mounted on a trolley to move the tables (Model MT TB)*
- *A movable base on castors (Model MT MB)*
- *The traditional base fixed to the floor (Model MT FB)*

The column base has been designed to allow an easy and immediate fitting of the surgical table top from several directions: sideways (from right and left), lengthways (from the head or feet) and by 45°. The column automatically recognises the direction of the table fitted.

The column has a console with display and membrane keyboard.

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The trolleys for moving the surgical table tops are made of stainless steel and have four swivel castors and practical handles for a safe grip. They are easy to use even in confined spaces and have all emergency movements, they are easy to clean and allow transportation of the surgical table tops with or without their base.

Three models are available:

- *Designed for fitting the table top from the head or from the feet end or by 45°. It has Trendelenburg and Reverse Trendelenburg movements (Model MT TL).*
- *Similar to the previous one, with variable height, it has automatic controls of the Trendelenburg and Reverse Trendelenburg movements and of the height adjustment (Model MT TLH).*
- *Trolley for fitting the table from the right or left side (Model MT L).*

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EXCELLENT PERFORMANCE TO MAKE OPERATING THEATRE WORK MORE EFFICIENT

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The Sordina X-25 patient transfer system, used in synergy with the MT System, makes it easy and fast to move the patient both before and after the operation. The patient can be moved from the bed or the stretcher to the operating table top, and back, without having to be lifted manually, meaning that the patient remains in the same position without any jerky movements.

Sordina X-25 improves hygiene and safety for patient and staff alike, speeding up transfer times and therefore making the operating theatre more efficient.

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The MT System permit excellent surgery organisation, improving the use of both time and space: in fact, downtime between one operation and the other is highly reduced.

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MT System allows fast, simple and precise surgical table top transfer. The patient is moved to the operating theatre in total safety: simply press a button, which has a buzzer and a flashing signal to inform the staff when the manoeuvre has been completed, to transfer the table top onto the column ready for surgery. Just one operator is sufficient to transfer the table top.

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MT System is compatible with all water jet washing systems. The MT System columns have a dedicated procedure to permit complete disinfection. The MT TB and MT MB columns can also be wheeled out of the operating theatre on the trolleys, to allow cleaning the theatre at the end of surgery.

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After surgery, the patient is transferred from the operating theatre quickly and safely with the MT System: simply press a button, which has a buzzer and flashing signal to inform the staff when the manoeuvre has been completed, and the table top can be transferred to the trolley.

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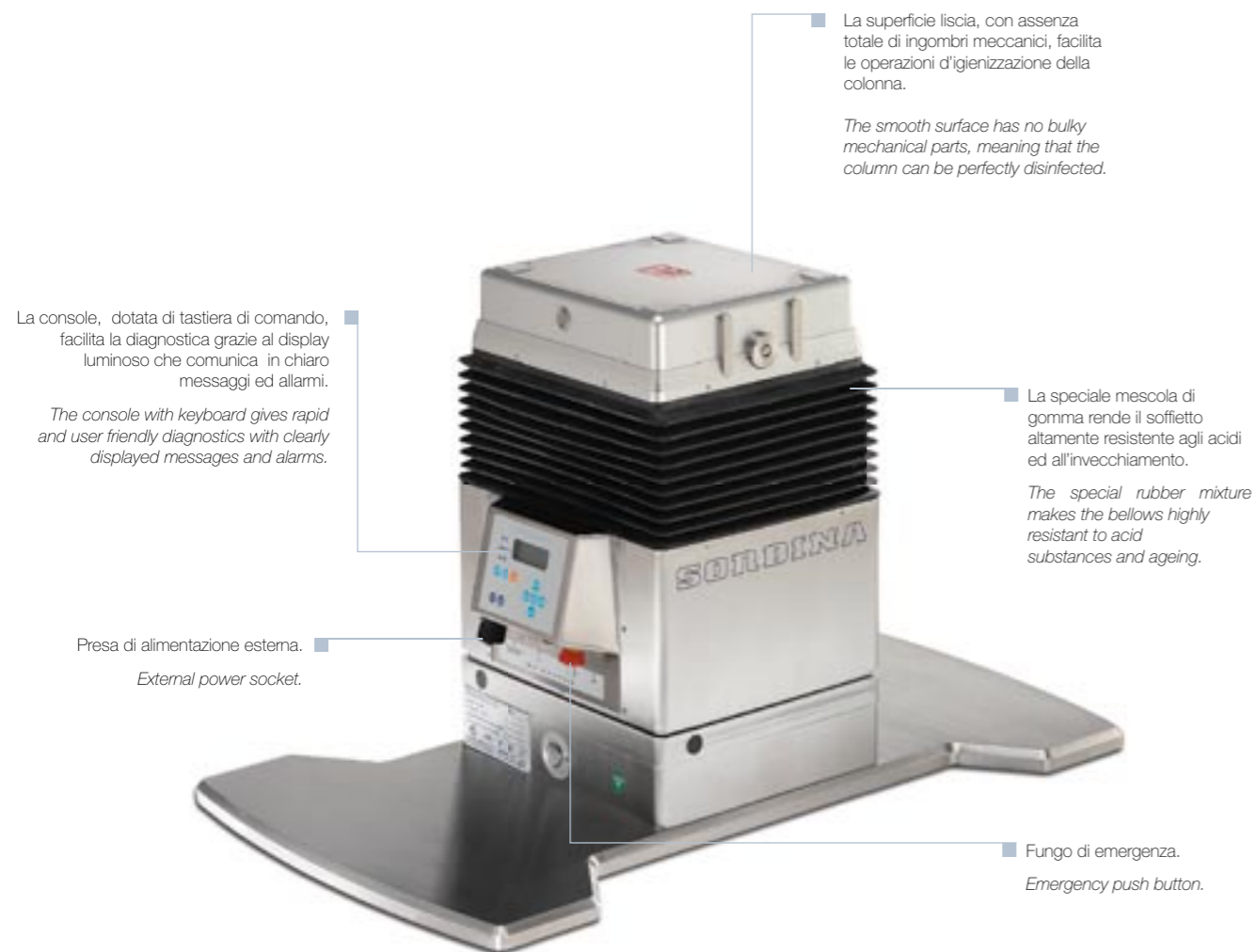
Simply press a button in order to transfer the table top or the complete operating table.

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VERSATILE BASES FOR ANY REQUIREMENT

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Advanced accessories for a forefront system

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The connection of the communication serial port to a PC helps maintenance work and permits setting the configuration parameters and displaying the status of the equipment.

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Technological design and special devices to make cleaning easier, with a dedicated procedure that is automatically activated to permit complete disinfection of the column.

Base piatta trasportabile mediante carrello per trasporto piani (Modello MT TB)
Flat movable base which is mounted on a trolley to move the table tops (Model MT TB)

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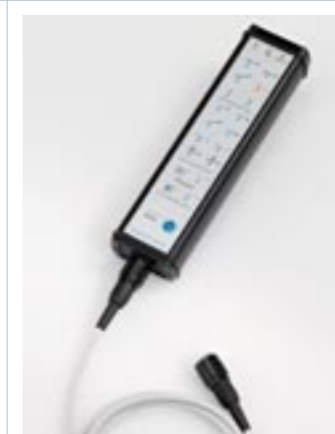
TROLLEYS FOR FAST AND SAFE TRANSFER

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Extremely easy to handle, the MT System rapid fitting trolleys are designed to transport only the surgical table top or the complete operating table system.

The trolleys have an automatic centring system on the column base, which is activated by the proximity sensors installed on the column.

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Height adjustment (800 ÷ 1000 mm) and Trendelenburg and Reverse Trendelenburg movements (+/-19°) are electrically powered by a cabled keyboard (Model MT TLH).

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Four double wheels (125 mm diameter) are made of special materials and adequate dimensions to prevent leaving any traces on the antistatic floor in operating theatres.

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A pedal lever allows controlling the direction of the wheels and applying the brakes.

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Con movimento Trendelenburg/Controtrendelenburg manuale ± 19° (Modello MT TL).
A trolley with lengthways movement for fitting and removing the table top, in the head-feet or feet-head direction, and at 45°.
Manual Trendelenburg and Reverse Trendelenburg (+/- 19°) movements (Model MT TL).

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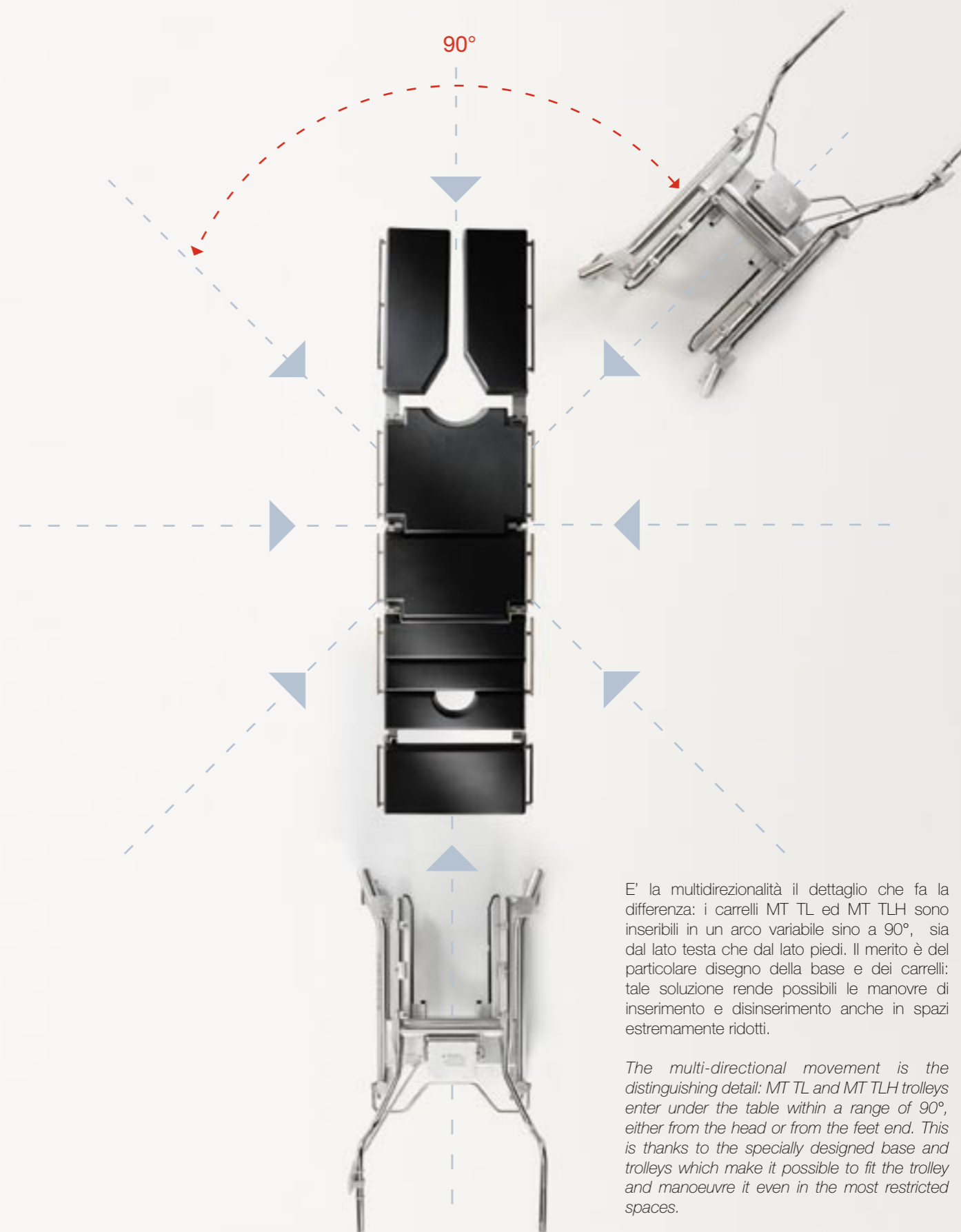
The exclusive advantages of the MT SYSTEM multi-directional trolleys

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The multi-directional movement is the distinguishing detail: MT TL and MT TLH trolleys enter under the table within a range of 90°, either from the head or from the feet end. This is thanks to the specially designed base and trolleys which make it possible to fit the trolley and manoeuvre it even in the most restricted spaces.

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*MT8 OPERATING TABLE TOP,
all the functions in a single table top*

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MT 8 is a unique multidisciplinary operating table top both for its features and for its performance. It can be composed by 6 to 11 section depending on the configuration, all made of stainless materials that are completely antistatic and radiotranslucent, unique and extremely original for the simple fast manner with which they can be interchanged. MT 8 has a virtually unlimited range of positions for the patient in order to satisfy all the needs of modern surgery.

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AUTOMATIC MOVEMENTS CONTROLLED THROUGH AN IR REMOTE CONTROL, CABLED KEYBOARD OR THE EMERGENCY CONSOLE

- minimum height 700 mm
- maximum height 1150 mm
- trendelenburg 40°
- reverse trendelenburg 40°
- lateral tilting right 25°
- lateral tilting left 25°
- inclination pelvis section -40°/40°
- inclination back section -20°/70°
- inclination upper back section -60°/30°
- inclination legs section -90°/30°
- translation 305 mm
- each single movement can be zero set by simultaneously pressing the respective movement buttons
- the entire table top can be zero set (automatic "0" of the table) by simply pressing the "Level" button
- the surgical table top can be automatically transferred from the trolley to the column and vice versa by simply pressing a button
- the entire operating table can be automatically transferred from the floor to the trolley and vice versa by simply pressing a button.

MANUAL MOVEMENTS

- inclination head section with double articulation large/narrow -60°/45°
- second inclination head section with double articulation large/narrow +45°
- inclination head section with single articulation large/narrow -60°/45°
- variation in the back section length +120 mm
- divarication legs table 0°-90°
- inclination lower legs sections with four sections -90°/0°
- breach for access of the surgeon leg sections with double compass movement 400 mm

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Double and single articulation head sections have manual servo-assisted gas spring movement.

The head section movements can be automated by fitting the section in the place of the upper back section or of the legs section.

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A single upper back section that can be extended according to specific requirements. Both the upper back and the lower back section have automatic movements.

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The MT8 surgical table top can be configured with the orthopaedics accessory for traction of the upper and lower limbs, mounted on a trolley (Model MT ORTO with MT ORTO TROLLEY).

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The MT8 table top can be configured with the back table formed by three sections, two of which can be removed for shoulder surgery.

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Among the various possibilities offered by the MT8, there is also the configuration for neonatal surgery.

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The width of the surgical table top can be increased by adding the optional extensions.

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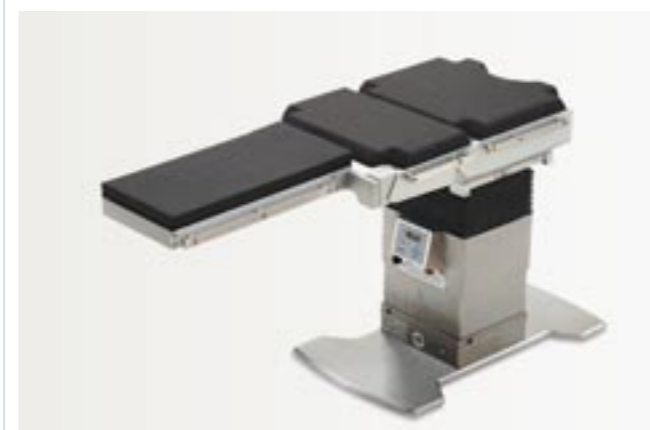
The additional section can be fitted at the head or at the feet end of the table.

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Legs table with support for automatic movements of the leg holders (MT S)

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The legs tables have automatic movements and can be fitted from the head or from the feet end of the table.

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UROLOGY POSITION
WITH AUTOMATIC MOVEMENT
OF THE LEG HOLDERS

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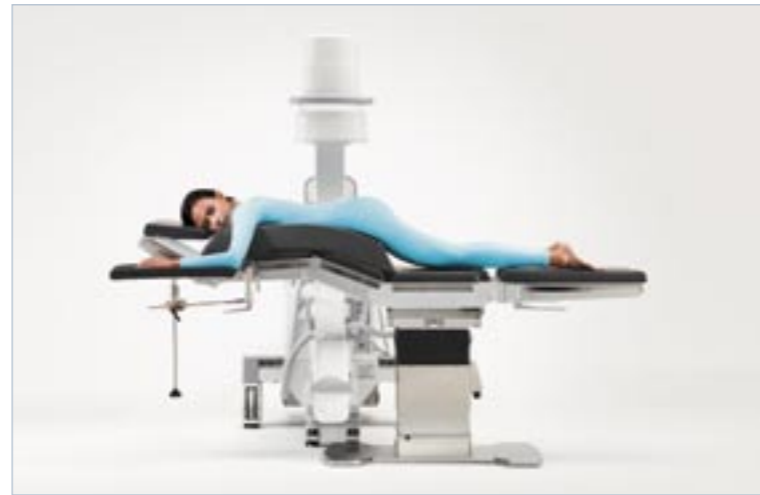
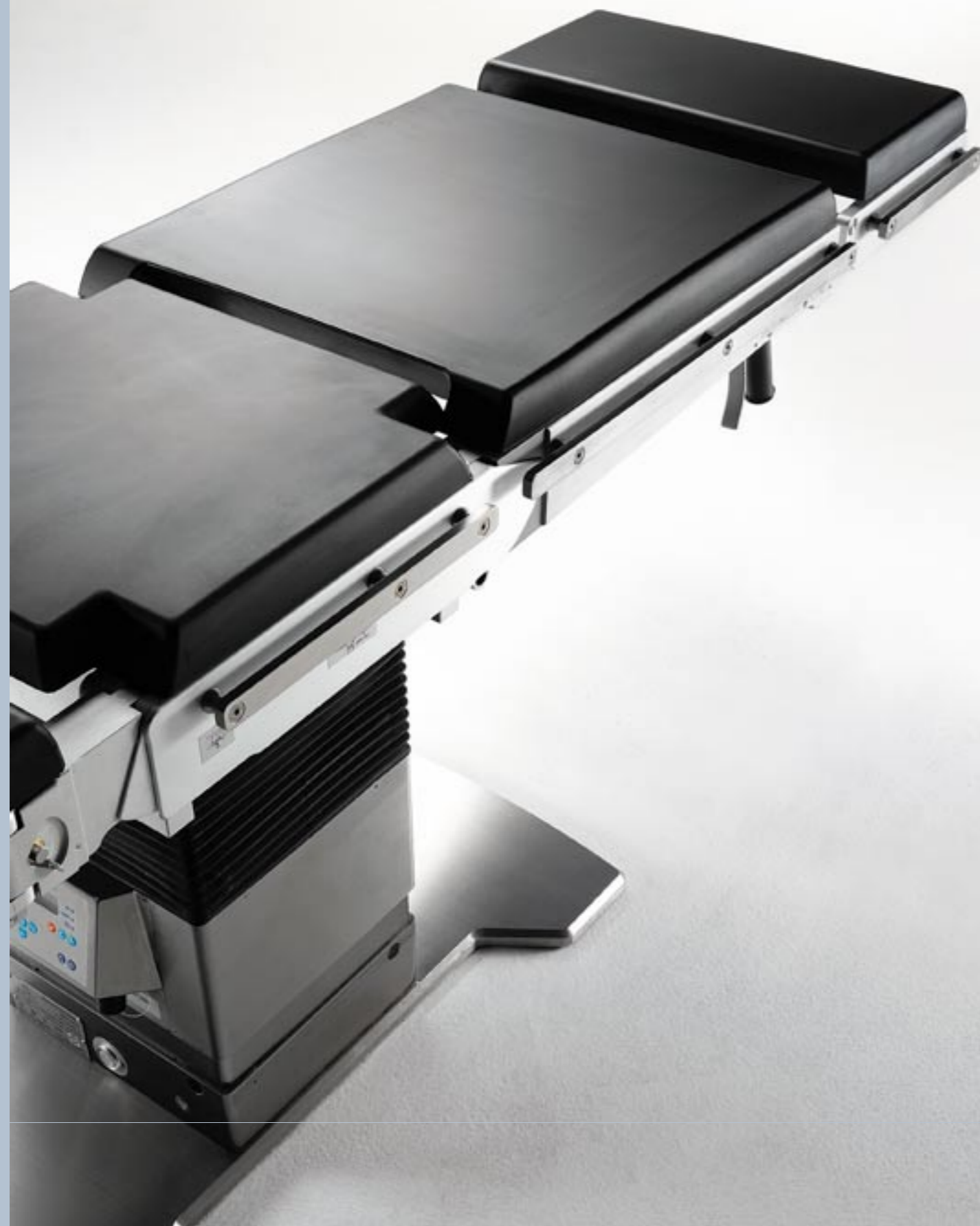
*MT6 OPERATING TABLE TOP,
many functions in a single table top*

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MT6 is a multidisciplinary operating table top which can be composed by several sections, from 4 to 7 according to the chosen configuration. Made of completely stainless materials, the sections are totally antistatic and radiotranslucent. Due to its multiple possibilities of configuration, the MT6 table top satisfies all the requirements of the numerous surgical disciplines.

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- minimum height 700 mm
- maximum height 1150 mm
- trendelenburg 40°
- reverse trendelenburg 40°
- lateral tilting right 25°
- lateral tilting left 25°
- inclination pelvis section -40°/40°
- each single movement can be zero set simultaneously pressing the respective movement buttons
- the entire table top can be zero set (automatic "0" of the table) by simply pressing the "Level" button
- the surgical table top can be automatically transferred from the trolley to the column and vice versa by simply pressing a button
- the entire operating table can be automatically transferred from the floor to the trolley and vice versa by simply pressing a button.

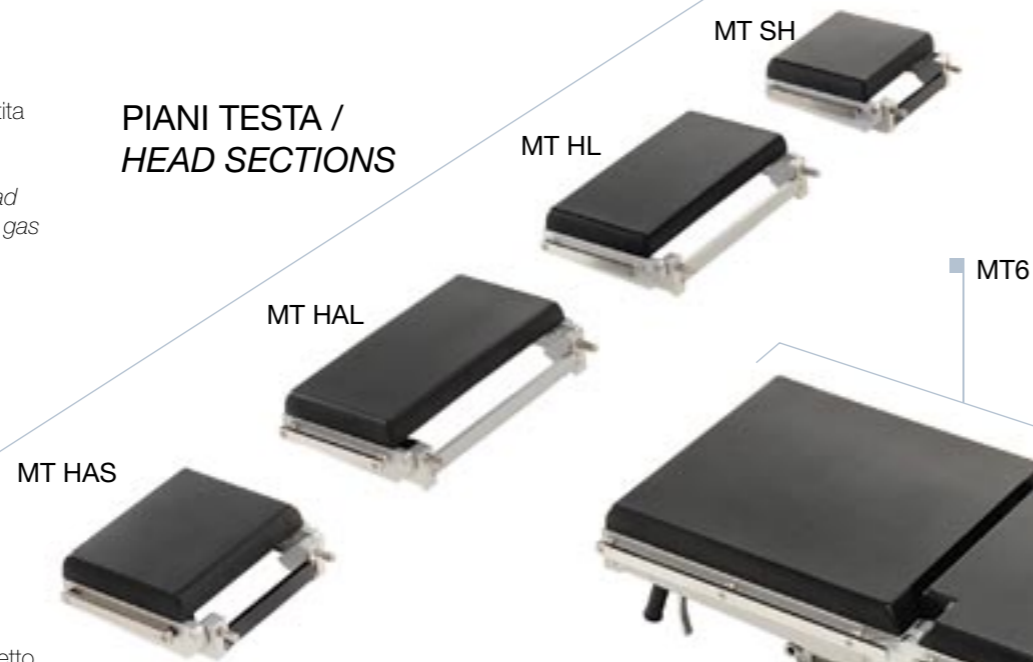
MANUAL MOVEMENTS

- back section inclination -20°/70°
- legs table inclination -90°/30°
- divarication legs table 0°-90°
- inclination lower legs sections with four sections -90°/0°
- breach for access of the surgeon leg sections with double compass movement 400 mm
- translation 305 mm
- inclination head table with double articulation large/narrow -60°/45°
- second inclination head table with double articulation large/narrow +45°
- inclination head section whit single articulation -60°/45°

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The double and single articulated head sections have manual servo-assisted gas spring movements.

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The MT6 surgical table top can be configured with the orthopaedics accessory for traction of the upper and lower limbs, mounted on a trolley (Model MT ORTO with MT ORTO TROLLEY).

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The MT6 table top can be configured with the back table formed by three sections, two of which can be removed for shoulder surgery.

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The MT6 table top can be configured for any sort of surgery, by adding the accessories that are supplied for the various disciplines.

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The back section has servo-assisted gas spring movements.

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MT6, MULTIPLE POSSIBILITIES FOR ANY KIND OF SURGERY

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The technical assistance of Sordina guarantees emergency responses at the shortest possible time, thanks to the numerous authorized technical centres.

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It can also offer a programmed maintenance service highly specialized.

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- *All Sordina products are designed and manufactured in conformity with the EEC directive 93/42/EEC.*
- *The Sordina Quality Assurance System is certified in conformity with UNI EN ISO 9001 and UNI EN ISO 13485 standards.*
- *The MT System operating tables are also certified by the IMQ mark label as further guarantee of their quality and safety.*
- *The accessories have been specially studied by the Sordina R&D department in close cooperation with important clinics, and the spare parts are all submitted to the company quality control procedures and are covered by warranty.*
- *Finally, to guarantee even further the high quality of Sordina products, we have our specialised, professional and expert technicians who are backed up by constant internal training programmes and our lengthy and unequalled traditions. A more than a century old tradition that is now combined with the ability of the new dynamic management, to give an increasingly careful and individual response to all the different needs of our current and future customers.*